# DeltaWash 10

Guida Rapida Utente  
User Quick Start Guide  
Guide rapide utilisateur  
Benutzer-Kurzanleitung  
Guía rápida Usuario  
Szybki Przewodnik Użytkownika  
Kullanıcı Hızlı Başlatma  
Hurtigstart For Bruker Rychlé  
Užívateľský návod  
Vlugge Start  
Γρήγορος Οδηγός Χρήστη  
Инструкция Для Быстрого Применения  
Guia Rápido do Utilizador  
Snabbstart för användare  
Käyttäjän pikaopas  
الدليل السريع للمستخدم  
Gyors használati útmutató  
Кратко ръководство на потребителя  
Kvikguide  
Trumposios instrukcijos  
Ghid de utilizare rapidă  
Užívateľská príručka  
Короткий посібник користувача

CE 0051

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Ref.	Device Description
1	Control panel and LCD display
2	Chamber filters
3	Door
4	Door closing system
5	Air inlet
6	Rotary nozzle
7	Stainless steel tank
8	HEPA filter
9	Stainless steel basket
10	Detergent
11	Salt
12	Detergent inlet
13	Decalcified water inlet
14	Temperature probe inlet
15	Door
16	USB port
17	Power switch
18	Inspection hole

Ref.	Description of electrical and hydraulic connections
19	Power cable
20	Mains fuses
21	Ethernet cable connection
22	Cooling fan
23	Foot
24	Hose for connection to the mains water with "Aquastop" system
25	Water drain connection
26	Air/steam outlet
27	Connection to external filling accessories

**NOTE**

UPON FIRST USE, IT IS RECOMMENDED TO RUN A MANUAL RESIN REGENERATION CYCLE FOLLOWED BY AN EMPTY 90° D DISINFECTION CYCLE.

**NOTE**

AFTER A PERIOD OF INACTIVITY EXCEEDING 24 HOURS IT IS NECESSARY TO CARRY OUT AN EMPTY 90° D DISINFECTION CYCLE BEFORE USING THE DEVICE.

CYCLE DESCRIPTION	°C WASHING TEMPERATURE	WASHING TIME	ULTRASOUND TIME	°C DISINFECTION TEMPERATURE	A0	DISINFECTION TIME	DRYING TIME	TOTAL CYCLE TIME	MAX TOTAL MASS (kg)
D90 DISINFECTION	45	6'	7'	90	3,000	4'	0' 6' + 25'	36'	2
W WASHING	45	6'	7'	-	-	-	0' 6' + 25'	20'	
PRE-WASHING	25	5'	-	-	-	-	-	7'	
D1 CUSTOM DISINFECTION	45	6' ÷ 15'	7' ÷ 16'	80 90	600 ÷ 1,200 (at 80°C) 600 ÷ 6,000 (at 90°C)	10'-20' (at 80°C) 1'-9' (at 90°C)	0 6' + 25'	According to settings	
W1 CUSTOM WASHING	45	6' ÷ 15'	7' ÷ 16'	-	-	-	0' 6' + 25'		
H HANDPIECES (*)	45	6'	-	90	3.000	4'	0' 6' + 25'	34'	

(\*) The H HANDPIECES cycle is available only with the HMD accessory; for the activation of the H HANDPIECES cycle contact the Service Centre.

#### WARNING

THE PRODUCT IS INTENDED EXCLUSIVELY FOR THE WASHING AND THERMAL DISINFECTION OF REUSABLE SURGICAL INSTRUMENTS CAPABLE OF WITHSTANDING A TEMPERATURE OF AT LEAST 80° C, WITH OR WITHOUT ULTRASONIC TREATMENT DEPENDING ON THE CYCLE.

IT IS INTENDED FOR NON-ULTRASONIC CLEANING AND DISINFECTION OF THE EXTERNAL SURFACES OF ROTATING INSTRUMENTS ONLY WHEN USED IN COMBINATION WITH THE APPROPRIATE ACCESSORY UNLESS SUCH INSTRUMENTS ARE EXPLICITLY DESIGNATED AS NOT SUITABLE FOR THERMAL DISINFECTION AND CLEANING BY THE MANUFACTURER.

IT IS NOT INTENDED FOR THE CLEANING AND THERMAL DISINFECTION OF THERMOLABILE MEDICAL DEVICES.

IT IS NOT INTENDED FOR THE CLEANING AND DISINFECTION OF INTERNAL DUCTS OF REUSABLE SURGICAL INSTRUMENTS THAT HAVE DEEP CAVITIES WITH REDUCED LIGHT.

IT IS NOT INTENDED FOR CLEANING AND/OR DISINFECTION OF ELECTRONIC INSTRUMENTS UNLESS EXPLICITLY INDICATED AS SUITABLE FOR THERMAL DISINFECTION AND CLEANING BY THE MANUFACTURER.

IT IS NOT INTENDED FOR WASHING INSTRUMENTS MADE OF UNTREATED ALUMINIUM OR WITH DECORATIVE ANODISING.

#### NOTE

TO ACHIEVE THE BEST PERFORMANCE OF THE DEVICE, REGULAR MAINTENANCE MUST BE CARRIED OUT ON A REGULAR BASIS.

PERIODICALLY A MESSAGE WILL BE DISPLAYED REQUESTING THAT THE SPECIFIED MAINTENANCE CHECKS BE PERFORMED.

FOR FURTHER INFORMATION OR IN CASE OF DOUBT, CONTACT THE TECHNICAL SERVICE.

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